


INSTRUCTION MANUAL Foot Massager

HER-FA500W

CONTENTS

PAGE

IMPORTANT SAFETY INSTRUCTIONS	· 1 ~ 4
NAME/FUNCTION OF EACH PART	···· 5 ~ 6
PREPARATIONS PRIOR TO USE	······ 7
HOW TO USE	·········· 8 ~ 11
AFTER USE	·········· 12
MAINTENANCE PROCEDURE	······ 12
TROUBLE-SHOOTING	·········· 13
SPECIFICATIONS	·········· 14
WARRANTY	·········· 15

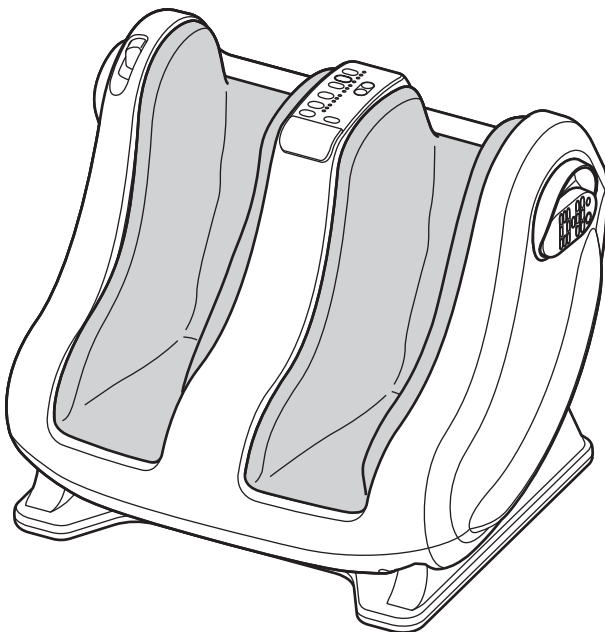

Be sure to read this manual thoroughly in order to ensure proper use. In particular, make certain to read the section "IMPORTANT SAFETY INSTRUCTIONS."
Keep this manual in a safe place so that it can be found easily.

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

Read all instructions before using this appliance.

DANGER

To reduce the risk of electric shock:

Always unplug this appliance from the electrical outlet immediately after using and before cleaning.

WARNING

To reduce the risk of burns, fire, electric shock, or injury to persons:

Handling the cord on this product or cords associated with accessories sold with this product, will expose you to lead, a chemical known to the State of California to cause cancer, and birth defects or other reproductive harm. **Wash hands after handling.**

An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.

Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.

Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons. Keep children away from extended back, foot support (or other similar parts).

Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.

Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.

Do not carry this appliance by supply cord or use cord as a handle.

Keep the cord away from heated surfaces.

Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.

Never drop or insert any object into any opening.

Do not use outdoors.

Do not operate where aerosol (spray) products are being used or where oxygen is being administered

To disconnect, turn all controls to the off position, then remove plug from outlet.

Connect this appliance to a properly grounded outlet only. See Grounding Instructions.

Use heated surfaces carefully . May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

Those who are receiving medical treatment or to whom the following categories apply should consult a doctor before using the massager.

- Those who use a medical electronic device implanted into the body (e.g. a pacemaker)
- Those who have malignant tumors
- Those who suffer from heart trouble
- Those who have lost temperature sensitivity
- Those who are menstruating, pregnant, or in the post-delivery period
- Those who suffer from sensory disturbance caused by an advanced peripheral circulatory disturbance such as diabetes
- Those who have skin lesions
- Those who require complete rest
- Those who have a body temperature of 38°C (100°F) or more (at the fever stage)
Example: those who are suffering from strong acute inflammation symptoms (fatigue, chill, blood pressure change, etc.), and from weakness
- Those who suffer from osteoporosis, and from acute painful conditions such as fracture of spine, sprain, pulled muscle, etc.
- Those who desire to use the massager on a part of the body that is diseased or receiving medical treatment.

Otherwise, it may cause an accident, injury or other health problems.

**All of the following information is important.
Be sure to follow the instructions carefully.**

Do not use the massager at any voltage other than the designated voltage.
Otherwise, it may cause fire or electric shock.

Prior to use, be sure that the cover cloth is not torn. If a tear is found, however small, stop using the massager immediately, unplug the power source, and arrange repair.
Otherwise, it may result in injury or electric shock.

Be sure to check that there is no-one, any pets or obstacles around the massager prior to reclining the massager.
Otherwise, it may result in injury.

After use, be sure to disconnect the power plug from the outlet and set the massager in "locked" mode.
Otherwise, children may bring about an accident, or injury may occur.

Do not allow persons in the following categories to use the massager.
Otherwise, they may be burned.

- **Those who have a diminished sense of temperature or pain**
- **Those who cannot move their own legs by themselves**

Avoid use by those who are prohibited from using massager by a physician.
Example: those who are suffering from thrombosis, severe aneurysm, acute varicose veins, various inflammation of skin, infectious skin disease (including subcutaneous inflammation), etc.

Avoid use by those who are unable to clearly indicate their intentions, as well as by physically disabled people.
Otherwise, it may result in accident or injury.

Do not allow children to use the massager. Also, while in operation, keep children away from the massager.
Otherwise, it may result in accident or injury.


Because the massager is for massaging your sole and calf, do not use it on the head, chest, abdomen, or other parts (knee, instep of foot, elbow, etc.)
Otherwise, the user may feel sick or an accident or injury may occur.

Do not stand on or sit on the massager.
Otherwise, it may result in failure of the massager or an accident or injury.

Be careful not to put your hand or foot into any clearance spaces.
Otherwise, it may result in injury.

Do not place your foot on the massager with excessive force.
Otherwise, it may cause failure of the massager or an accident or injury.

Do not place your feet in the manner shown in the figure.
Otherwise, it may cause an accident or injury.

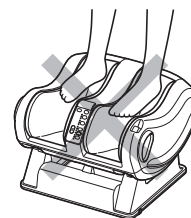

Do not use for a prolonged period. It may result in low temperature burns.

Do not damage, break, modify, bend forcibly, pull, twist or bundle the cord. In addition, do not put heavy objects on the cord or jam the cord between objects.
Otherwise the cord may break, resulting in fire or electric shock.

Be careful not to use the massager when the cord or the power plug is damaged or the plug socket is loose.
Otherwise, it may cause an electric shock, short circuit or fire.

SAFETY PRECAUTIONS

Do not wind the cord around the massager.

Otherwise, unreasonable force may be applied to the cord, possibly resulting in fire or electric shock.

Never attempt to dismantle the massager. Do not allow any personnel other than a qualified repair engineer to dismantle or repair the massager.

Otherwise, it may cause a fire, electric shock, accident or injury.

Contact the sales agent if repairs are required. Be sure to disconnect the power plug from the outlet prior to maintenance. Also, do not attempt to plug in/unplug the power plug with wet hands.

Otherwise, it may result in electric shock, accident or injury.

Prior to use, be sure to check if switches, etc, are functioning normally according to this Instruction Manual.

Push the power plug into the outlet securely.

Otherwise, it may cause an electric shock, short circuit or fire.

If discomfort or pain is felt during use, stop using the massager and consult a doctor immediately.

If the user continues to use the massager in such a way, an unexpected accident or injury may occur.

Aim to use the massager for up to 15 minutes at a time.

The continuous use of the massager for an extended period may result in excessive stimulation, and may result in adverse effects or injury.

If the massager becomes inoperative, disconnect the power plug immediately to prevent accidents, and consult the sales agent for inspection/repair information.

Otherwise, it may result in fire due to electric shock, current leakage, short circuit, etc.

Do not attempt to disconnect the power plug by pulling the cord, but always by holding the power plug and pulling it out. Also, clean the dust off the power plug periodically.

Otherwise, it may result in electric shock, short circuit or fire.

When you have not used the massager for a long period of time, check if it is operating normally and safely prior to use. Also, check that there is no damage to the cord, etc.

Otherwise, it may result in an accident or injury.

After use, remove dirt from the massager, and neatly store in a damp-free area.

Otherwise, accumulated dust or dampness may cause fire, electric shock or failure of the massager.

Do not use for any other purpose other than massaging your soles or calves.

Otherwise, it may result in the failure of the massager, or an accident or injury.

Do not hang up or swing the massager around by gripping the cord. Also, do not stand the massager on the side with the cord underneath.

It may result in disconnection of the cord.

Do not spill water or any other liquid on the massager, or subject it to a strong impact.

Otherwise, it may cause a fire, electric shock or failure of the massager. Stop using the massager immediately and consult the sales agent for inspection.

Do not use the massager with medical instruments.

Otherwise, the user may feel sick, or it may cause other health problems.

Do not use the massager by placing it inside bedding or covering it with a blanket etc.

Otherwise, it may cause a fire or high temperature inside the massager.

Do not put your finger in the groove where the shiatsu part moves.

Otherwise, it may cause an accident or injury.

Do not allow any person whose shoe size is less than 21cm(8 1/4") to use the massager.

Otherwise, it may cause an accident or injury.

Do not use the massager by setting foot size to larger than the size of your foot.

Otherwise, it may cause an accident or injury.

**All of the following information is important.
Be sure to follow the instructions carefully.**

Do not use the massager in the bathroom, or places with high humidity.
Otherwise, it may cause a fire, electric shock or failure of the massager.

When the equipment is not going to be used for an extended period, be sure to disconnect the power plug from the outlet.

Otherwise, it may cause an electric shock, current leakage or fire due to damaged insulation.

If there is a power failure, pull the power plug from the outlet immediately.

Otherwise, it may result in an accident or injury when the power is restored.

Burns will result from improper use read instructions before using do not wet-do not use pins never remove cover.

To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

SAVE THESE INSTRUCTIONS

Grounding Instructions

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock.


This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

DANGER

Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product – if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

This product is for use on a nominal 120-volt circuit, and has a grounding plug that looks like the plug illustrated in sketch A in following figure. A temporary adapter that looks like the adapter illustrated in sketch B and C may be used to connect this plug to a 2-pole receptacle as shown in sketch B if a properly grounded outlet is not available.


The temporary adapter should be used only until a properly grounded outlet (sketch A) can be installed by a qualified electrician. The green colored rigid ear, lug, or the like extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by a metal screw.

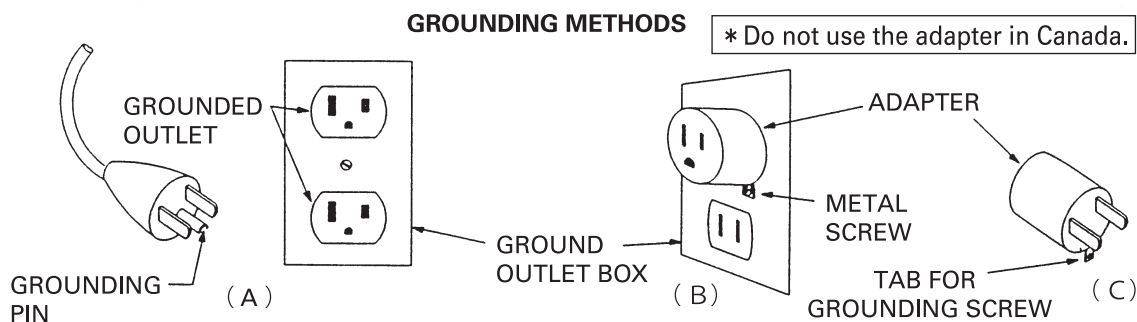

This appliance is intended for Household Use Only.

NAME/FUNCTION OF EACH PART

MAIN UNIT

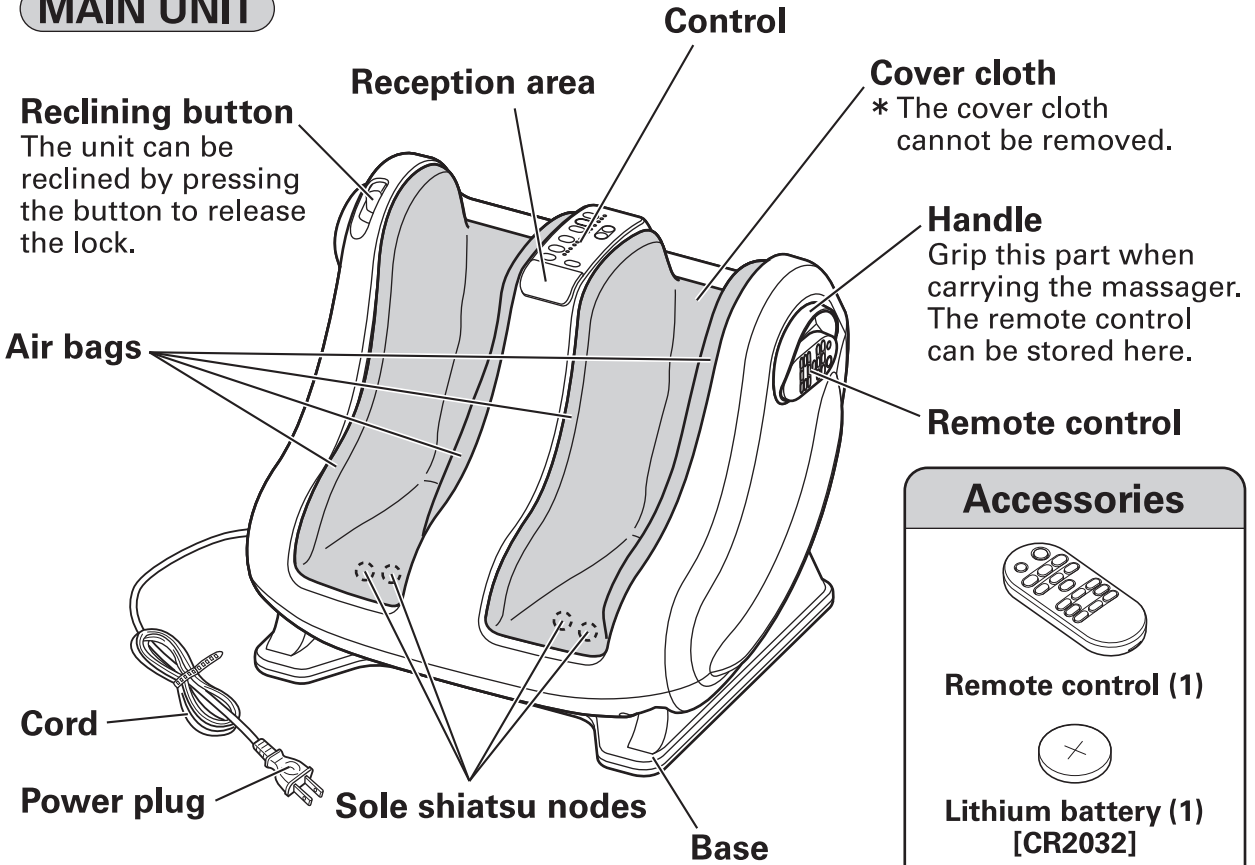

Accessories




Remote control (1)



Lithium battery (1)
[CR2032]

The battery is included when the massager is shipped. Its life may be shortened due to self-discharge.


CONTROL

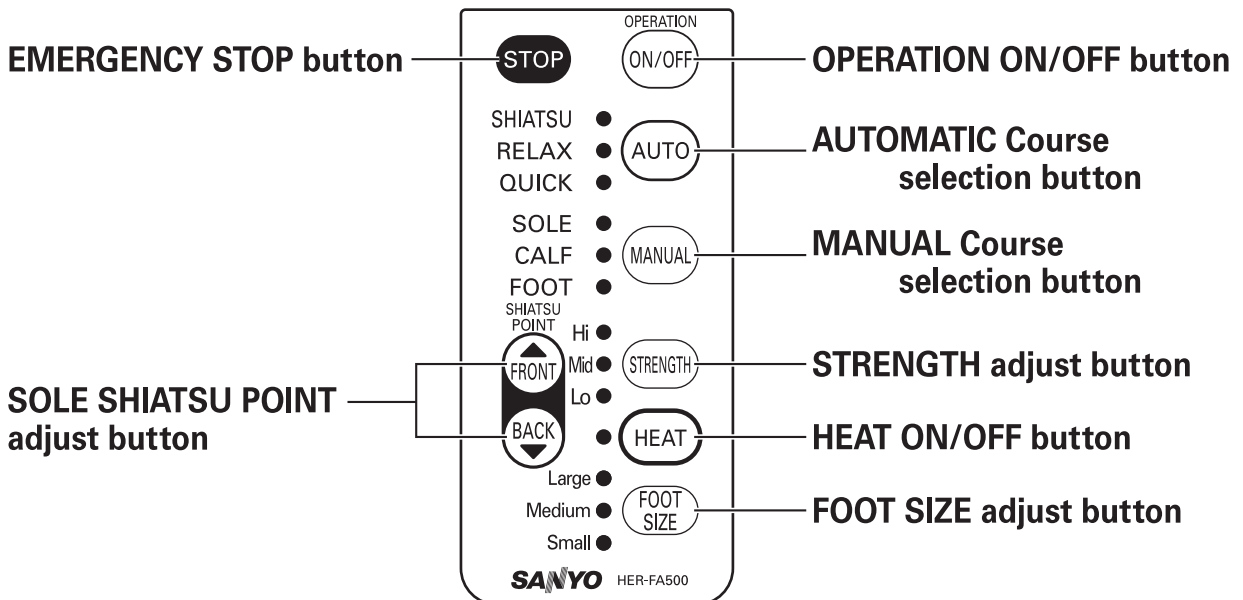


REMOTE CONTROL

EMERGENCY STOP button

Transmission point

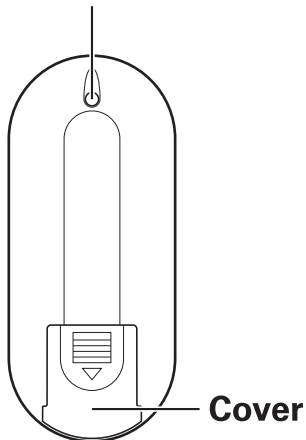

〈Back side〉

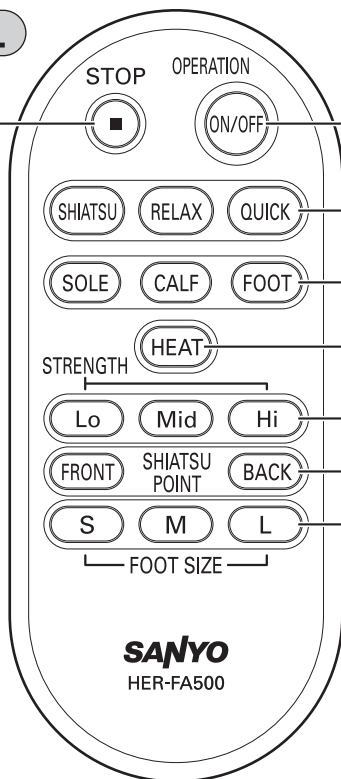

OPERATION ON/OFF button

AUTOMATIC Course selection button

MANUAL Course selection button

HEAT ON/OFF button

STRENGTH adjust button


SOLE SHIATSU POINT adjust button

FOOT SIZE adjust button

Insertion/removal of lithium battery


Inserting battery

① Slide the cover by pressing it.

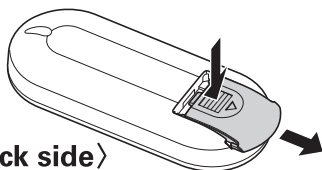

〈Back side〉

② Insert the battery securely by tilting it.

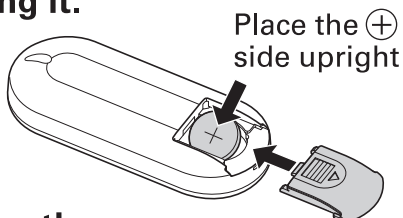


Place the ⊕ side upright

③ Close the cover.

Removing battery

● Insert a thin stick (non metal) into the hole, lift up the battery to remove it.

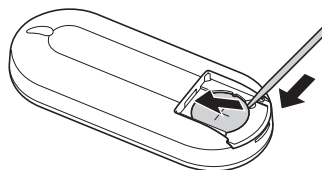

NOTE

- Please make sure ⊕ and ⊖ sides of the battery are correctly inserted.
- Use [CR2032] lithium battery.
- Replace the battery with a new one when the remote control does not operate smoothly.

CAUTION

Keep the lithium battery out of the reach of children. If swallowed, please consult a doctor immediately.

PREPARATIONS PRIOR TO USE

1 Decide on the place to use the massager.

- Place the massager on a level floor surface.
- Keep the massager at least 1 meter away from AV appliances like TV or radio. Otherwise, the program may be disturbed by noise.


2 Check the cover cloth.

- Prior to use, be sure that cover cloth is not torn. If a tear is found, however small, stop using the massager immediately, unplug from the power source, and send it for repair. Otherwise, it may result in injury or electric shock.

3 Connect the power plug to the outlet.


4 Place your feet in the massager.

- Place your heels on the rear part of the massager as shown in the figure.
- Press the Reclining button to release the lock, and recline the massager to the desired angle.
- When the massager is upright it is in "locked" mode. (It does not lock other than in this position.)

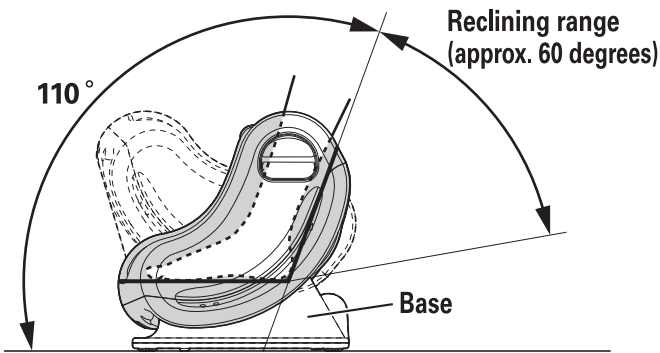

WARNING

Do not place your feet in the manner shown in the figure. Otherwise, it may cause an accident or injury.

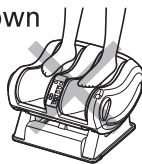


NOTE

- Do not recline the massager before pressing the button. Otherwise, unreasonable force may be applied to the massager and it may cause a failure of the massager.

Example of use


When you sit on a chair

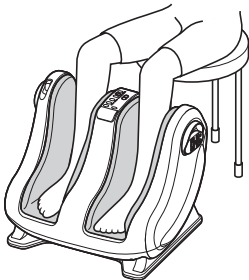


NOTE

- Do not sit on a chair with casters when using the massager. Otherwise, it may result in injury.

When you sit on a sofa

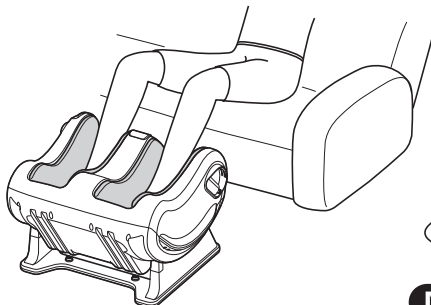

When you lie on the floor

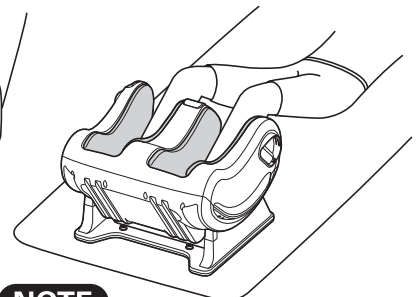

NOTE

- Use with your knees slightly bent.


HOW TO USE

How to use Remote Control



Press the button while aiming the transmission point on the back of the Remote Control in the direction of the reception area of the Main Unit.

- Reception range of the Main Unit is 20 degrees back and front, left and right from the front (center) of the reception area. The further from the front (center), the harder it is to achieve reception.

Transmission point

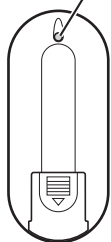

〈Back side of Remote Control〉

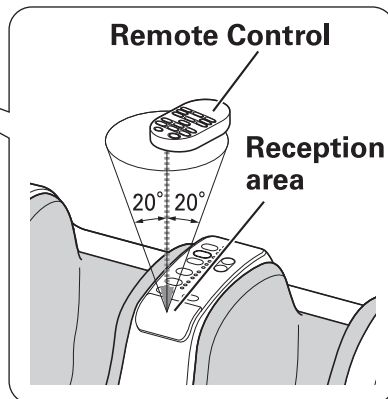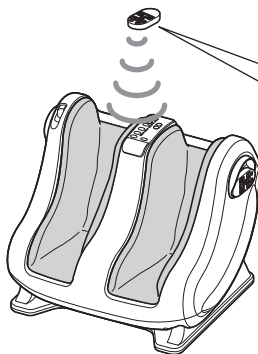


NOTE

- Do not cover the transmission point and reception area with hands or objects. The Remote Control may not operate.
- Remote Control may not operate in a room where receiving part is in direct sunlight, or where inverter lighting equipment/electronic instant operation lighting equipment is in use.
- Do not step on, drop, or allow water to contact the Remote Control. It may cause damage or failure.
- Remote Control of this massager may activate other foot massagers with remote controls. Exercise caution when other foot massagers are in use.

When Remote Control is not in use, store it in the handle.


* It can be attached on left or right handle.

Face the Remote Control buttons to the outside to attach to the Handle.

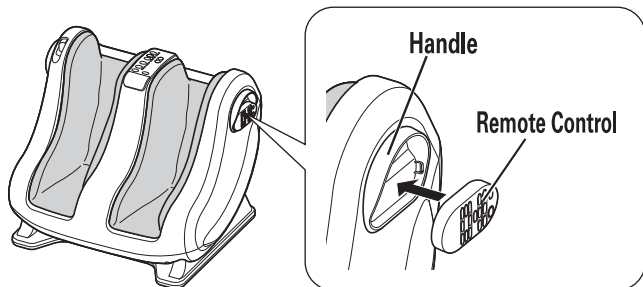

NOTE

- Do not try to attach it with the buttons inside. It may result in damage or failure.

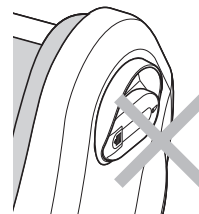

〈Incorrect storage position〉

HOW TO USE

1 Press the OPERATION ON/OFF button.

- When you press the button, the 「Mid」 part of the strength lamp, the heat lamp, and the 「23.5-26.5」 of the foot size lamp will light up, and the heater will turn on.
- When the sole shiatsu nodes do not touch the toes, it will move to the toe position and stop. (Calf lamp blinks while moving. Only the EMERGENCY STOP button can be activated while blinking.)



2 Turn 「ON」 or 「OFF」 the HEAT (heater).

- Every time you press the HEAT ON/OFF button, the 「ON」 and 「OFF」 mode of the heater changes. (When the heater is 「ON」, the lamp on the Main Unit illuminates.)
 - Turn 「OFF」 the heater if you feel your soles are hot.
- *When the room temperature is low such as in winter, it may take a long time to heat up or you may not feel warm. Warming the massager before operating it is recommended.



3 Choose the foot size by pressing FOOT SIZE adjust button.

- Massage is performed according to the selected foot size. (Refer to the size of your shoes)

	FOOT SIZE		
	S	M	L
cm	21.0-23.0	23.5-26.5	27.0 +
UK	2-4	4½-8	8½+
EUR	34½-37	37½-43	43½+
US Mens	N/A	6-9½	10+
US Womens	5-6½	7-11	11½+

Note: Foot size conversions are approximate.



- Every time you press the button, the foot size changes following the arrow in the figure, which is displayed by the lamp.



- Press the FOOT SIZE adjust button of your choice. Lamp on Main Unit is illuminated.

NOTE

- Please do not change the foot size during the massage. Be sure to stop massaging when switching the foot size.

4 Press the desired massaging button.

AUTOMATIC COURSES

Main Unit



- When you press the desired AUTOMATIC Course selection button, the operation mode changes following the arrow in the figure below, which is displayed by the lamp on the Main Unit.



Remote Control



- Each time the AUTOMATIC Course selection button is pressed, the operation starts, which is displayed by the lamp on the Main Unit.
- Massage stops when the operation button in action is pressed again.

SHIATSU	Substantial massage, mainly shiatsu. To massage the soles, sides of your feet and calves automatically. (approx. 15 min.)
RELAX	A comfortable and gentler massage. To massage the soles, sides of your feet and calves automatically. (approx. 15 min.)
QUICK	Short massage, mainly shiatsu. To massage the soles, sides of your feet and calves automatically. (approx. 5 min.)

MANUAL COURSES

Main Unit



- Each time the MANUAL Course selection button is pressed, the operation mode changes following the arrow in the figure below, which is displayed by the lamp on the Main Unit.



Remote Control



- When you press the desired MANUAL Course selection button, the operation starts, which is displayed by the lamp on the Main Unit.
- Massage stops when the operation button in action is pressed again.

SOLE	To massage soles and the sides of your feet. (approx. 15 min.)
CALF	To massage calves. (approx. 15 min.)
FOOT	To massage the sides of your feet and calves. (approx. 15 min.)

HOW TO USE

5 Adjust the strength of massage and the position of the sole shiatsu nodes.

STRENGTH adjustment

Main Unit



- Each time the STRENGTH adjust button is pressed, the strength of massage changes following the arrow in the figure to the right, which is displayed by the lamp on the Main Unit.



Remote Control



- The massage strength changes when you press the desired STRENGTH adjust button, which is displayed by the lamp on the Main Unit.

SOLE SHIATSU POINT adjustment

* You can adjust the shiatsu position during manual operation.

- While the FRONT button is being pressed, the sole shiatsu nodes move forward.
- While the BACK button is being pressed, the sole shiatsu nodes move backward.

Main Unit




Remote Control

■ Finishes massaging automatically by timer operation.

- The massaging will stop automatically after approx. 15 minutes (5 minutes in Quick course). The sole shiatsu nodes move to the toe position and stop. (With the 「beep」 alarm sound, all the lamps on the Main Unit are turned off.) When the massage is over, the heater turns off simultaneously.

To stop the massage during massaging operation.

- Press the OPERATION ON/OFF button. Sole shiatsu nodes move to the toe position, and stop. (CALF lamp blinks while moving. All the lamps on the Main Unit are turned off after movement is finished. Only the EMERGENCY STOP button can be activated before the movement is finished.)

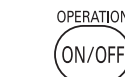

Main Unit

OPERATION



Remote Control

When you feel sick, or in emergency situation

- Press EMERGENCY STOP button. Massage stops immediately, and all the lamps on the Main Unit are turned off.
- When resuming operation, repeat from 1 on page 9.



Main Unit

STOP



Remote Control

When the safety device has worked

- If unreasonable force is applied to the sole shiatsu nodes the operation will stop. (The CALF lamp blinks, and the Lo-Hi lamps light up.) In that case, pull the power plug out of the outlet, and after three seconds or more, reinsert the plug into the outlet. You can use the massager again.

When massage is too strong or painful

Insert a towel between your feet and the massaging part of the Main Unit to ease the strength.

AFTER USE

WARNING


Be sure to disconnect the power plug from the outlet.
Put the massager in 「locked」 mode.

CAUTION

When the equipment is not going to be used for an extended period, be sure to disconnect the power plug from the outlet.

■ Return the massager to its original position, and into the 「locked」 mode.

- Massager is in 「locked」 mode when the lock clicks.

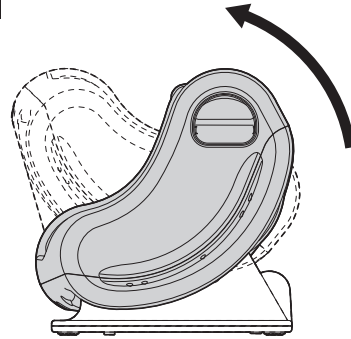


MAINTENANCE PROCEDURE


WARNING

Be sure to disconnect the power plug from the outlet prior to maintenance. Also, do not attempt to plug/unplug the power plug with wet hands.

1 Soak a soft fabric in diluted mild kitchen detergent, and wring it out.

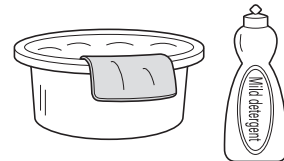

2 Wipe the massager with the fabric, and rub it with a dry cloth when finished.

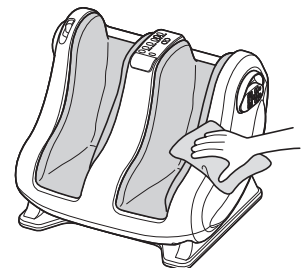

NOTE

- Do not attempt to use benzene, thinner, bleach or alkaline detergent for cleaning, as this may result in deformation, discoloration or cracking.

TROUBLE-SHOOTING

WARNING

- If the trouble still persists even after the following proper countermeasures have been taken, stop using the massager to avoid possible accidents.
Disconnect the power plug from the outlet, and consult the sales agent.
- Do not attempt to repair the equipment at home by yourself, as it is very dangerous.

Type of Trouble	Check points	Countermeasure
Unusual sound during operation	<ul style="list-style-type: none"> • These sounds may be unavoidably generated due to construction, but they will not effect the function, etc. <ul style="list-style-type: none"> ▪ Sound of air massage "Pusshu" sound (Change valve sound) Hissing sound (Exhaust sound) "Buu" sound (Sound of pump operation) "Pako" sound (Sound when the air bag fills) 	<ul style="list-style-type: none"> • Shiatsu operating sound, rolling back and forth switching operation sound of Sole Shiatsu ("clunk" sound) • Rolling back and forth switching operation sound of Sole Shiatsu ("Ween Ween, Schul Schul" sound)
Operation failure	<ul style="list-style-type: none"> • Check that the power plug is securely plugged in the outlet . 	<ul style="list-style-type: none"> • Push the power plug into the outlet securely. (See page 7)
	<ul style="list-style-type: none"> • Is SHIATSU lamp flashing and the Lo-Hi lamps lit? (When inserting the power plug into the wall outlet, the contact may not be steady) 	<ul style="list-style-type: none"> • Re-insert the power plug into the wall outlet.
Remote Control failure	<ul style="list-style-type: none"> • Battery is discharged. 	<ul style="list-style-type: none"> • Replace battery. (See page 6)
	<ul style="list-style-type: none"> • Battery is inserted incorrectly. (⊕ and ⊖) 	<ul style="list-style-type: none"> • Insert in the correct way. (See page 6)
	<ul style="list-style-type: none"> • Transmission point is covered with fingers. 	<ul style="list-style-type: none"> • Do not cover the transmission point with your fingers. (See page 8)
	<ul style="list-style-type: none"> • Transmission point of Remote Control is not aimed at the reception area of Main Unit. 	<ul style="list-style-type: none"> • Aim transmission point of Remote Control at reception area of Main Unit. (See page 8)
	<ul style="list-style-type: none"> • There is a barrier between transmission point of Remote Control and reception area of Main Unit. 	<ul style="list-style-type: none"> • Remove the barrier. (See page 8)
The operation stops in the middle of massaging	<ul style="list-style-type: none"> • This phenomenon is caused by the operation of the timer, but is not a problem. 	
	<ul style="list-style-type: none"> • Check if unreasonable force has been applied to the sole shiatsu nodes. (The CALF lamp blinks, and the Lo-Hi lamps light up.) 	<ul style="list-style-type: none"> • The safety device of the massager is working. (See page 11)
Sole does not feel warm	<ul style="list-style-type: none"> • When the room temperature is low such as in winter, it may take a long time to heat up or you may not feel warm. Warming the massager before operating it is recommended. (See page 9) 	

SPECIFICATIONS

Power consumption	51 W
Rated electricity of the heater	36 W
Timer	Automatic courses: 15 minutes (Quick course is 5 minutes) Manual courses: 15 minutes
Dimensions	550 mm(21 3/4'') [width] × 445 mm(17 5/8'') [depth] × 435 mm(17 1/8'') [height]
Weight	13.5 kg (30 lbs.)
Fabric material	Polyester
Length of cord	2 m (79'')

SANYO FOOT MASSAGER LIMITED WARRANTY

(USA ONLY)

OBLIGATIONS

In order to obtain warranty service, the product must be delivered to and picked up from an Authorized SANYO Factory Service Center at the user's expense, unless specifically stated otherwise in this warranty.

If you need assistance with product operation, authorized service center referral, service assistance, problem resolution, or parts and accessories availability; you may contact us by:

- A. Visiting www.sanyoservice.com on a 24 hr/7 days a week basis or
- B. Calling the customer support center at 1-800-421-5013
Weekdays 7:30 AM- 5:00 PM Pacific Time

THIS WARRANTY IS VALID ONLY ON SANYO PRODUCTS PURCHASED AND USED IN THE UNITED STATES OF AMERICA. THIS WARRANTY APPLIES ONLY TO THE ORIGINAL RETAIL USER AND DOES NOT APPLY TO PRODUCTS USED FOR ANY INDUSTRIAL, PROFESSIONAL OR COMMERCIAL PURPOSE. THE ORIGINAL DATED BILL OF SALE OR SALES SLIP MUST BE SUBMITTED TO THE AUTHORIZED SANYO SERVICE CENTER AT THE TIME WARRANTY SERVICE IS REQUESTED.

Subject to the OBLIGATIONS above and EXCLUSIONS below, SANYO FISHER COMPANY (SFC) warrants this SANYO products against defects in materials and workmanship for the periods of LABOR and PARTS specified below. SFC will repair or replace (at its option) the product and any of its parts which fail to conform to this warranty.

The warranty period commences on the date the product was first purchased at retail.

LABOR	PARTS
1 YEAR	1 YEAR

EXCLUSIONS

This warranty does not cover (A) the adjustment of customer-operated controls as explained in the appropriate model's instruction manual, or (B) the repair of any product whose serial number has been altered, defaced or removed.

This warranty shall not apply to the following items: non-electrical / mechanical attachments; accessories; damage to head cover, back pad, arm covers, seat cover, foot rest cover when not caused by defects in workmanship.

This warranty shall not apply to setup, installation, removal of the product for repair or reinstallation of the product after repair.

This warranty does not apply to repairs or replacements necessitated by any cause beyond the control of SFC including, but not limited to, any malfunction, defect or failure caused by or resulting from unauthorized service or parts, improper maintenance, operation contrary to furnished instructions, shipping or transit accidents, modification or repair by the user, abuse, misuse, neglect, accident, incorrect power line voltage, fire, flood or other Acts of God, or normal wear and tear.

The foregoing is in lieu of all other expressed warranties and SFC does not assume or authorize any party to assume for it any other obligation or liability.

THE DURATION OF ANY WARRANTIES WHICH MAY BE IMPLIED BY LAW (INCLUDING THE WARRANTIES OF MERCHANTABILITY AND FITNESS) IS LIMITED TO THE TERM OF THIS WARRANTY. IN NO EVENT SHALL SFC BE LIABLE FOR SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING FROM OWNERSHIP OR USE OF THIS PRODUCT, OR FOR ANY DELAY IN THE PERFORMANCE OF ITS OBLIGATIONS UNDER THIS WARRANTY DUE TO CAUSES BEYOND ITS CONTROL.

SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS AND/OR DO NOT ALLOW THE EXCLUSION OR LIMITATION OF CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS AND EXCLUSIONS MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS. YOU MAY HAVE OTHER RIGHTS, WHICH VARY FROM STATE TO STATE.

For your protection in the event of theft or loss of this product, please fill in the information below for your own personal records.

Model No. _____ Serial No. _____
(Located on back or bottom side of unit)

Date of Purchase _____ Purchase Price _____

Where Purchased _____

Please register your foot massager at
www.sanyomassagechairs.com



SANYO Fisher Company
21605 Plummer Street
Chatsworth, CA 91311
www.sanyousa.com

9BF-6-P111-341AE-1 0908